

Changing dermatology forever Unprecedented innovation in laser technology





Unprecedented innovation in laser technology

With fractional laser resurfacing the laser beam is broken up or fractionated into many small micro beams which are separated so that when they strike the skin surface small areas of the skin between the beams are not hit by the laser and left intact. These small areas of untreatedskin promote a much more rapid recovery and healing with less risk of complications. The small areas treated by the fractional micro beams, called micro treatment zones, cause sufficient laser injury to promote new collagen production and resultant facial skin rejuvenation.

Acne scars can be treated with the Fractional CO_2 laser in order to help resurface and recontour atrophic scars of the face, chest, and back. The fractional CO_2 laser has been used effectively in many countries to resurface acne scars and post-traumatic scars of the face and body.


How Fractional Resurfacing works | Epidemia | Papsins | | Demis | Papsins | | Demis | Resoular | | Demis | Demis | Demis | | Demis | Demis | Demis | | Resoular | Demis | Demis | Demis | | Resoular | Demis | Demis | Demis | | Resoular | Demis | Demis | Demis | | Resoular | Demis | Demis | Demis | Demis | | Resoular | Demis | Demis | Demis | | Resoular | Demis | Demis | Demis | | Resoular | Demis | Demis | Demis | | Resoular | Demis | Demis | Demis | | Resoular | Demis | Demis | Demis | Demis | | Resoular | Demis | Demis | Demis | Demis | | Resoular | Demis | Demis | Demis | Demis | | Resoular | Demis | Demis | Demis | Demis | Demis | | Resoular | Demis | Demis | Demis | Demis | Demis | Demis | | Resoular | Demis | | Resoular | Demis | Demis





Application:

- 1. Remove all kinds of wrinkles (brow veins, glabellas veins, neck veins, pregnancy veins and so on)
- 2. Strengthen, tightening& whitening skin, face promotion, Skin rejuvenation, face lift
- 3. Age spots and blemishes, acne and speckle treatment: blain acne scabby acne, allergic acne, papilla acne, improving the appearance of liquidic skin and acne pit.
- 4. Treat all kinds of telangiectasis, blood vessels, blood silks. 5. Repair of sun damaged skin on the face, neck, shoulders and hands
- 6. Remove all kinds of neoplasms, wart, syringoma, sebaceous gland tumor, becker nevus, haemangioma, adipoma, steatoma, etc.
- 7. Vaginal tightening, vaginal rejuvenation.

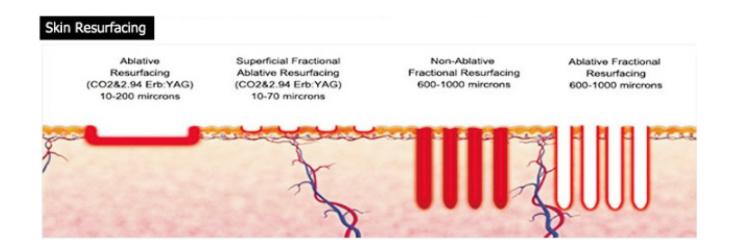

Feature:

Skin Rejuvenation, Scar, Acne Removal, Skin Resurfacing, cutting, vaginal tightening

Advantages:

- 1.30W RF laser tube
- 2. American coherent RF tube
- 3. Fractional, cutting, vaginal tightening 3 models
- 4. Korea imported sevem joints light guiding arm
- 5. Seven scanning shapes
- 6. Laser working mode: Continuous
- 7. No pain, no cut, no side-effects;
- 8. No rebound! Apparent long-last result;
- 9. Particular designed intelligent laser control system ensures a precise treatment;
- 10. RF-exited CO2 laser energy shooting is an even steady and safe treatment process;
- 11. The best result for Vaginal tightening vaginal rejuvenation.





BEFORE AFTER BEFORE AFTER

SPECIFICATION

(CO ₂ fractional laser machine
Nominal Wavelength	10600nm
Laser power	30W
Work mode	Fractional, cutting, vaginal tightening
Pulse width	pulse mode:0.1ms-100ms fractional mode:0.1ms-10ms
Beam delivery	seven joint Articulated Arm
Power	110V/220 V~ 50Hz



About PZ Laser Company

PZ Laser a worldwide aesthetic device manufacturer produces high-end products for plastic surgeons, dermatologist, physicians and healthcare professionals. PZ Laser serves the industry as one of the leading companies with rigorous R&D level and valuable experience with 15 years histories. PZ Laser products are being globally marketed by overseas offices and worldwide distributors partners.





Tel: 0086 - 371 - 55677868

0086 - 371 - 55677886 0086 - 371 - 55011989 Fax: 0086 - 371 - 55677886

E-mail: info@pzlaser.com

• Headquarters:

Building 13, Electronic Industrial Park, Hi-tech Development Zone, Zhengzhou, Henan, China

For more information: www.pzlaser.com